

Cybertech™

IMPORTANT TO KNOW

- The Pulley system requires that you place the handle next to power unit and have product fully open (cords extended) each time you put it on to maximize support.
- Placement around low back is critical for proper support.
- Power unit must be centered on spine when fully extended.
- Product can be worn over or between clothing. It should always be worn over at least a T-shirt.

Fitting

The fitting of the CYBERTECH™ can be done in five easy to follow steps:

1. Place handle as close to power unit as possible. The handle is typically on the left side of your body, unless you have been instructed otherwise.

2. Stand, make sure handle is next to power unit, hold the brace by the side panels horizontal to the floor.
3. Place the brace around your body being sure that the power unit is centered on the spine, and the cords are not twisted. Adjust the brace up or down on your torso to find the best position for support.
4. Once you have found the best location for the brace on your body, pull slightly outward on the side panels to maximize the open position. Close the front of the brace by velcroing the right side over the left.
5. Pull handle away from the velcro and across your body. Tighten the brace until desired support is achieved. Attach handle to the velcro strips located on the sides of your brace. Loosen the brace by removing handle from the velcro strips and reattaching it at a point that allows more comfort.

User Tips

- Pull handle gently at first, until you get used to the Power Unit.
- Tighten to stand, loosen to sit.
- While wearing the brace, practice normal daily activities and movements to get used to the brace.

- Pulling in stomach muscles before tightening makes pulling the handle easier.
- Do not wear the brace tight for extended periods of time unless instructed by a physician. Consult your physician for a complete wear schedule.
- This product was designed to enable adjustment throughout the day to provide increased support when you need it and less support when you don't.
- To disconnect the power unit, slide the power unit outwards on side panels. To reattach, reverse this process. The power unit slides on and off of metal clips.

Care

The CYBERTECH™ can be cleaned by hand washing the brace in cold water, drying with a towel, and then air drying the brace in a shady area. DO NOT put the brace in an automatic dryer.